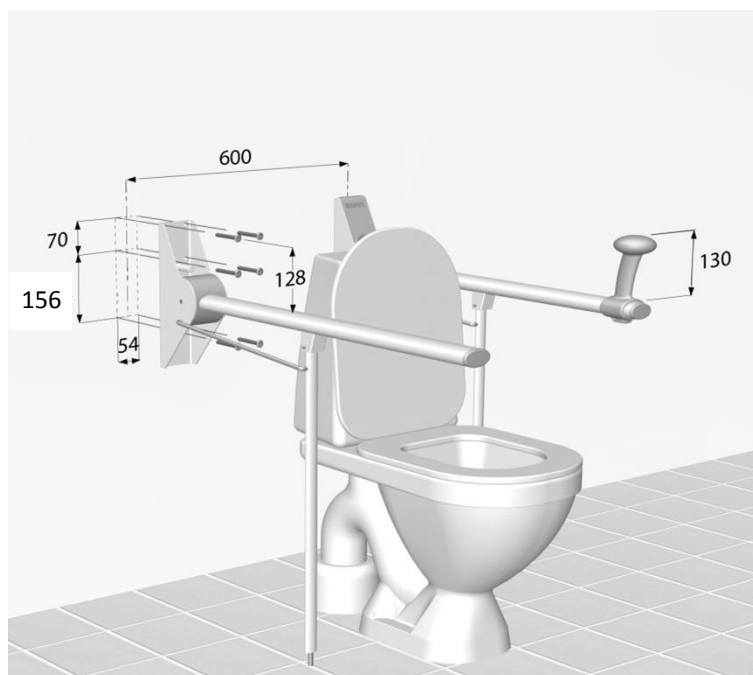
# Ropox Straight Toiletstøtte 135, 40-40120

/ Ropox Straight Toilet Support Arms 135

/ Ropox Straight Toilettenstütze 135

Montageanvisning / Mounting instruction / Montageanleitung

## I henhold til / according to DS 3028 (Danish Standard)

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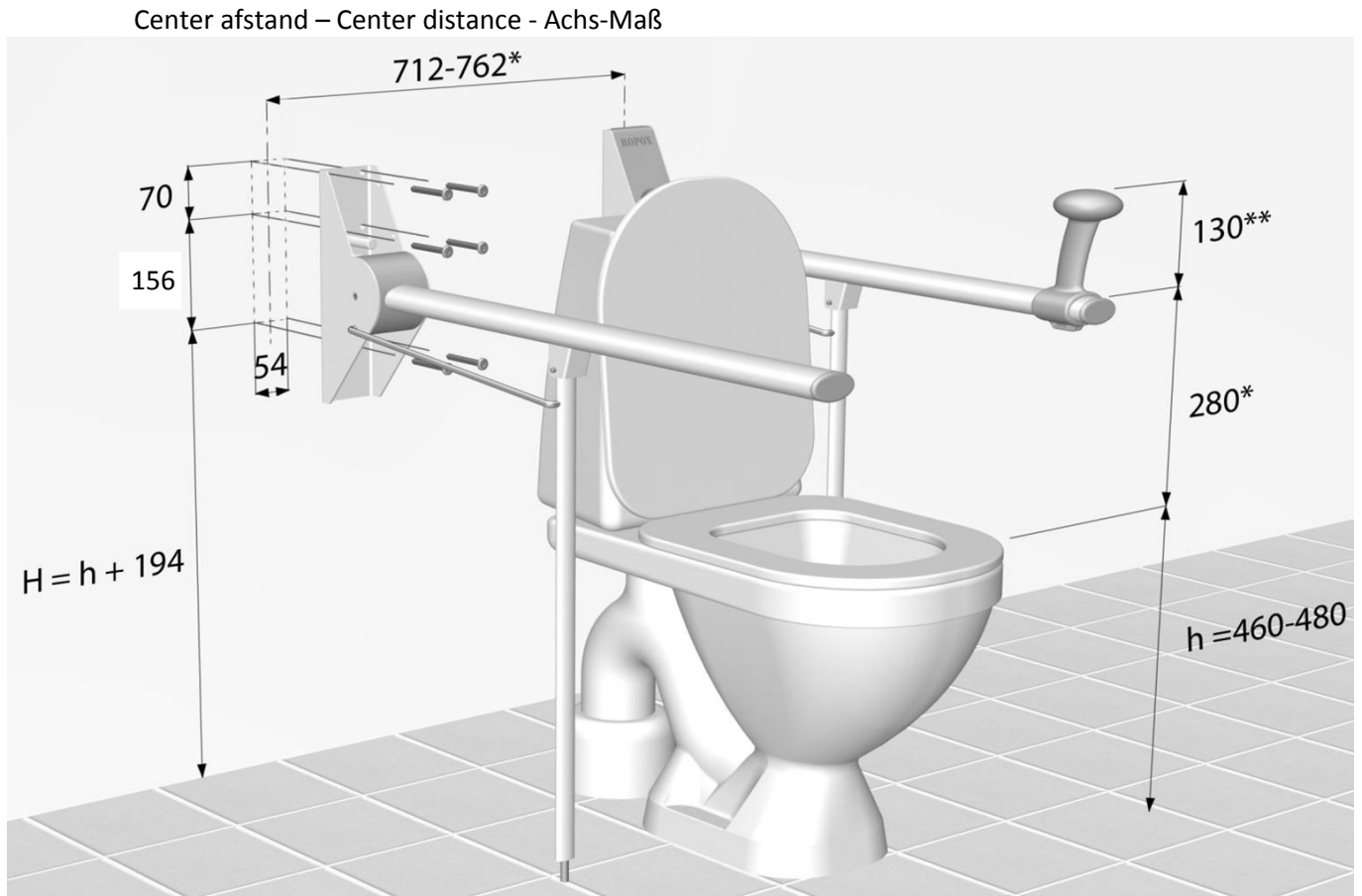
Please find attached mounting template showing position of the 6 holes for easy installation

### User heights and installation heights (upper holes in mounting template/bracket)

User height (recommendation)	Installation height (upper holes)	Situation
800 mm (top of support arm)	928 mm	Ropox Straight Toilet Support Arm
670 mm (top of support arm) 800 mm (top of Get up/sit down)	798 mm	Ropox Straight Toilet Support Arm + Get up/sit down fitting
<p>If any other specific user height is required, simply position the 2 upper holes of the bracket (or the mounting template) 128 mm above the required height of the support arm.</p> <p><u>Example:</u> required user height is 775 mm Installation height = 775 + 128 mm = 903 mm</p>		

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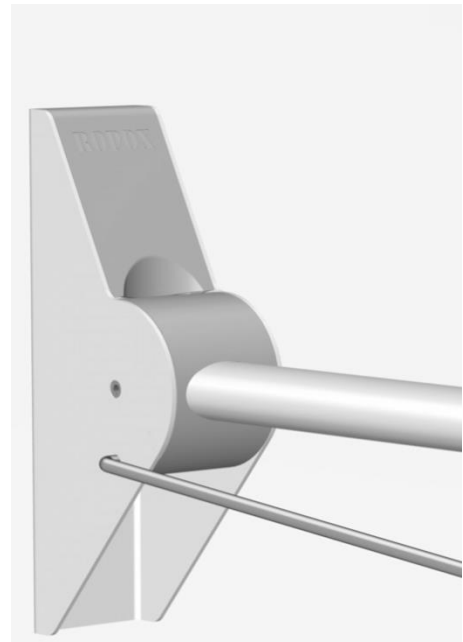
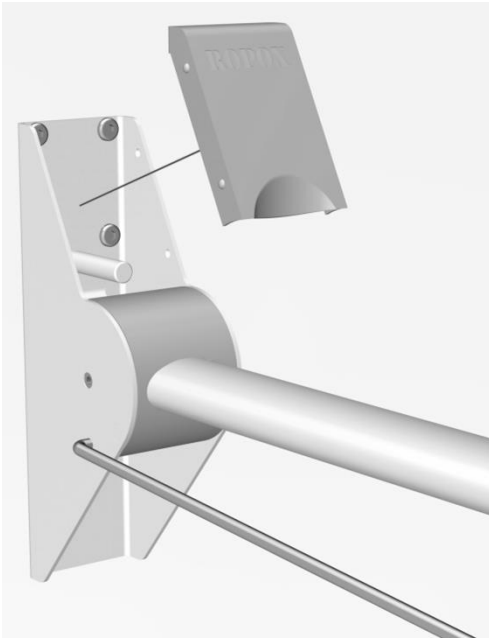
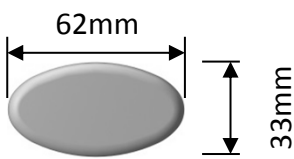
## Wall mounting

The screws and rawlplugs included are for mounting on a concrete wall. The installer must always examine the material, condition and strength of the wall and use appropriate screws and rawlplugs.

All Toilet Support Arms are tested and safe up to and including 400kg. To comply with DS/EN 12182 the maximum user load is set to 135kg.

Maximum pull in the top screws are 100kg.

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## Maintenance

The toilet supports are maintenance free. For safety reasons it is recommendable to check all mounting screws and moving parts to be sure all is tightened ok, and that all attachment to the wall is in correct and safe condition.

## Cleaning

To clean this product, do not use a high-pressure washer. This product is not machine washable.

Simply use a soft cloth or sponge with soapy water (pH 7) of max 60°C. Always clean this product after use to prevent dirt from sticking to the surface.

Important:

Do not use scouring powder or other abrasives as they may scratch the surface. Also avoid the use of cleaning utensils with sharp edges. Never use strong mineral acids or alkali.

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## CE-erklæring

Ropox hereby declares that the following product

40-40120                      Ropox Straight Toilet Support Arms 135

is classified as Class I and conforms to the following directive and standard:

### Directives

Directive No. 93/42/EEC concerning Medical Devices (risk class 1)

Machinery Directive No. 2006/42/EC, amended by Directive No. 98/37/EC.

### Standards

- DS/EN 12182: 2012:            Assistive devices for persons with functional disabilities -  
  General requirements and test methods
- DS/EN 14971: 2012:            Medical devices -  
  Application of risk management to medical devices
- DS/EN 9999: 2011:            Assistive devices for persons with functional disabilities -  
  Classification and terminology

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## **Complaints**

See General Terms of Sale and Delivery on [www.ropox.com](http://www.ropox.com)

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## ROPOX A/S